Labcorp Oklahoma, Inc. Test Directory

Myoglobin, Serum

Order Name: MYOGLOBIN
Test Number: 2004240
Revision Date: 12/12/2022

TEST NAME			METHODOLOGY	LOINC CODE
Myoglobin, Serum			Quantitative Electrochemiluminescent Im	nmunoassay 2639-3
SPECIMEN REQUIR	EMENTS			
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	1mL (0.2)	Serum	Clot Activator SST	Room Temperature
Instructions	Notes: 0.7 mL (Note: This volume Does NOT allow for repeat testing.) Specimen Type: Gel-barrier tube (preferred) or red-top tube Specimen Storage: Room temperature Specimen Collection: If red-top tube is used, transfer separated serum to a plastic transport tube. Special Instructions: This test may exhibit interference when sample is collected from a person who is consuming a supplement with a high dose of biotin (also termed as vitamin B7 or B8, vitamin H, or coenzyme R). It is recommended to ask all patients who may be indicated for this test about biotin supplementation. Patients should be cautioned to stop biotin consumption at least 72 hours prior to the collection of a sample. Specimen Stability: Ambient: 14 days, Refrigerated: 14 days, Frozen: 14 days			

GENERAL INFORMATION		
Expected TAT	1 - 3 days	
Performing Labcorp Test Code	010405	
Notes	Labcorp Test Code: 010405	
CPT Code(s)	83874	
Lab Section	Reference Lab	

Service provided by Labcorp Oklahoma, Inc. All Rights Reserved. © 2003 - 2025