

Meconium Drug Screen (5 Panel)

Order Name: **DRUG F SCR**

Test Number: 4066250

Revision Date: 12/20/2022

TEST NAME	METHODOLOGY	LOINC CODE
Meconium Drug Screen (5 Panel)	Immunoassay (IA)	

SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	5g (1g)	See Instructions	See Instructions	Refrigerated
Instructions	<b>CHAIN OF CUSTODY TEST !</b> <b>Specimen Type:</b> Meconium Collection Kit (PeopleSoft#96663) OR clean, leakproof polypropylene container. <b>Specimen Storage:</b> SUBMISSION/TRANSPORT (<3 days): Room Temperature. For storage beyond three days, specimen should be refrigerated or frozen. <b>Specimen Collection:</b> Specimens from different voidings may be pooled if necessary <b>Special Instructions:</b> Testing referred to MedTox Laboratories Inc MX#4034			

GENERAL INFORMATION	
Expected TAT	3-6 Days
Clinical Use	All fecal drug screens are analyzed by immunoassay. Each positive result is confirmed by GC/MS or LC/MS/MS to the limit of detection.  Meconium Drug Screen Confirmations
Performing Labcorp Test Code	809373
Notes	Labcorp Test Code: 809373
CPT Code(s)	80307
Lab Section	Reference Lab