## **Testosterone Total, Females and Children**

Order Name: **TESTO TO F**Test Number: 3610975
Revision Date: 03/20/2017

TEST NAME			METHODOLOGY	LOINC CODE	
Testosterone Total, Females and Children			Liquid Chromatography/Tandem Mass Spectron	metry 2986-8	
SPECIMEN REQU	JIREMENTS				
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment	
Preferred	1 mL (0.2mL)	Serum	Clot Activator SST	Refrigerated	
Alternate 1	1 mL (0.2mL)	Serum	Clot Activator (Red Top, No-Gel)	Refrigerated	
Instructions	'	Preferred to separate serum from cells ASAP or within 2 hours of collection.  Stability after separation from cells: Ambient: 24 hours; Refrigerated: 1 week; Frozen: 6 months.			

GENERAL INFORMATION		
Sun-Sat Sun-Sat		
2-5 Days		
This test is suggested for women and children 13 years and under due to an improved sensitivity of testosterone by LC-MS/MS.		
84403		
Reference Lab		

Service provided by Labcorp Oklahoma, Inc. All Rights Reserved. © 2003 - 2024