Labcorp Oklahoma, Inc. Test Directory

Mannose-binding Lectin (MBL) (Mannon)

Order Name: MANNOSE B L

Test Number: 6905069 Revision Date: 12/12/2022

TEST NAME Mannose-binding Lectin (MBL) (Mannon)			METHODOLOGY	LOINC CODE
			Quantitative Enzyme-Linked Immunosorb	ent Assay 30152-3
SPECIMEN REQU	JIREMENTS			
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	0.5 mL (0.1)	Serum	Clot Activator SST	Room Temperature
Instructions	Notes: 0.1 mL (Note: This volume Does NOT allow for repeat testing). Specimen Type: Gel-barrier tube is preferred. Red top tube is acceptable. Specimen Storage: Maintain specimen at room temperature. Stability: Room temperature up to 14 days Refrigerated up to 14 days Frozen up to 14 days Freeze/thaw cycle: stable x3 Specimen Collection: If red-top tube is used, transfer separated serum to a plastic transport tube.			

Specimen Stability: Ambient: Not Available, Refrigerated : Not Available, Frozen: Not Available

GENERAL INFORMATION		
Expected TAT	5 - 7 days	
Performing Labcorp Test Code	004900	
Notes	Labcorp Test Code: 004900	
CPT Code(s)	83520	
Lab Section	Reference Lab	

Service provided by Labcorp Oklahoma, Inc. All Rights Reserved. © 2003 - 2025