

Chlamydia trachomatis, Neisseria gonorrhoeae SurePath - TMA

Order Name: CT/GC SurePath

Test Number: 5560330

Revision Date: 08/15/2022

TEST NAME	METHODOLOGY	LOINC CODE
Neisseria gonorrhoeae Probe	Transcription-Mediated Amplification	50388-8
Chlamydia trachomatis Probe	Transcription-Mediated Amplification	50387-0
Specimen Source - Prompt		

SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	See Instructions	PAP specimen	SurePath Liquid Pap/APTIMA Transport Tube	Refrigerated
Instructions	Submit Cervical brush in SurePath liquid-based Pap test Media or Vortex SurePath media and transfer 1 mL to APTIMA Specimen Transfer Tube (ARUP supply #42711) within 24 hours of collection. To reduce the potential for contamination, SurePath specimens should be poured off, using sterile technique,into the APTIMA Specimen Transfer Tube prior to Cytology Testing. <b>Transport Refrigerated</b> Please mark collection and requisition as SurePath Collection <b>Unacceptable specimens:</b> Specimen FROM A URINE Aptima specimen Transfer Media. Specimens from patients that are less than 16 years of age. <b>Specimen source is required</b> <b>Stability:</b> SurePath Media: Ambient: 24 hours; Refrigerated: 48 hours; Frozen: 1 week <b>APTIMA Specimen Transfer Tube FROM PAP SUREPATH:</b> Ambient: 2 weeks; Refrigerated: 1 month; Frozen: 1 month			

GENERAL INFORMATION	
Testing Schedule	Sun-Sat
Expected TAT	1-4 Days after test set up
Notes	Reference Lab: ARUP Test Code: 2001551 <a href="#">Click Here</a> to view information on the ARUP website.
CPT Code(s)	87491; 87591
Lab Section	Reference Lab