

2,3 Dinor-11Beta-Prostaglandin F2 Alpha 24 Hour Urine

Order Name: 2,3Dinor-11b

Test Number: 6907123

Revision Date: 01/10/2023

TEST NAME	METHODOLOGY	LOINC CODE
2,3Dinor-11bPG F2a Ur	Liquid Chromatography/Mass Spectroscopy	94381-1
2,3Dn11bPG F2a/Creat	Liquid Chromatography/Mass Spectroscopy	
2,3Dn11bPG F2a Collection	Liquid Chromatography/Mass Spectroscopy	
2,3Dn11bPG F2a Tot Vol	Liquid Chromatography/Mass Spectroscopy	
2,3Dn11bPG F2a Creat Conc	Liquid Chromatography/Mass Spectroscopy	

SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	4 mL (3 mL)	Urine, 24-hour	Sterile Screwtop Container	Refrigerated
Instructions	<p>Patient Prep: Patients taking aspirin or nonsteroidal anti-inflammatory drugs (NSAIDs) may have decreased concentrations of prostaglandin F2 alpha. If possible, discontinue for 2 weeks or 72 hours, respectively, prior to collecting a specimen.</p> <p>Remarks: Collection duration and urine volume must be provided for testing. Total collection volume must be greater than 500 mL.</p> <p>Specimen Prep:24 hour urine. Refrigerate specimen during collection. Transfer 5mL (4mL) in urine pour off container. Submit refrigerated.</p> <p>Stability: Ambient - 8 hours; Refrigerated -14 days; Frozen -30 days</p> <p>Test is not performed at ARUP; separate specimens must be submitted when multiple tests are ordered</p>			

GENERAL INFORMATION	
Testing Schedule	Varies
Expected TAT	3-9 days after set-up
Notes	Reference Lab: ARUP Test Code: 3002337 Click Here to view information on the ARUP website.
CPT Code(s)	84150
Lab Section	Reference Lab