

Tularemia Antibody IgG/IgM

Order Name: **Tularem AB G/M**
Test Number: 5570900
Revision Date: 04/29/2024

TEST NAME	METHODOLOGY	LOINC CODE
Tularemia IgM	Enzyme-Linked Immunosorbent Assay (ELISA)	93716-9
Tularemia IgG	Enzyme-Linked Immunosorbent Assay (ELISA)	93717-7
Tularemia interpretation	Interpretive information.	93718-5

SPECIMEN REQUIREMENTS

Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	1 mL (0.5)	Serum	Clot Activator SST	Refrigerated or Frozen
Instructions	Allow specimen to clot completely at room temperature. Separate serum or plasma from cells ASAP or within 2 hours of collection. Specimen Stability: Room Temperature: Unacceptable, Refrigerated: 2 Weeks, Frozen at -15C; Unlimited.			

GENERAL INFORMATION

Testing Schedule	Mon - Fri
Expected TAT	1-2 Days
Clinical Use	Diagnosis of exposure to Francisella tularensis.
CPT Code(s)	86668x2
Lab Section	Immunology - Serology