Labcorp Oklahoma, Inc. Test Directory

Toxoplasma gondii DNA, Qualitative Real-Time PCR

Order Name: **TOXO PCR** Test Number: **5592225** Revision Date: **12/06/2022**

TEST NAME		METH	IODOLOGY	LOINC CODE	
Toxoplasma gondii DNA, Qualitative Real-Time PCR		Polyn	nerase Chain Reaction/Real Time		
SPECIMEN REQUIREMENTS					
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment	
Preferred	1 mL (0.2mL)	Whole Blood	EDTA (Lavender Top)	Refrigerated	
Alternate 1	1 mL (0.2mL)	Whole Blood	ACD Solution A or B (Yellow Top)	Refrigerated	
Alternate 2	1 mL (0.2mL)	CSF (Cerebrospinal Fluid)	Sterile Screwtop Container	Refrigerated	
Instructions	 Specimen Type: Sterile container (CSF), lavender-top (EDTA) tube, OR yellow-top (ACD) tube Specimen Storage: Whole blood is STABLE at room temperature or refrigerated for seven days; DO NOT freeze. CSF is STABLE at room temperature or refrigerated for seven days, FROZEN for 90 days. refrigerated for 14 day and FROZEN for 90 days. Specimen Collection: To avoid delays in turnaround time when requesting multiple tests on frozen samples, PLEASE SUBMIT SEPARATE FROZEN SPECIMENS FOR EACH TEST REQUESTED. Specimen Stability: Ambient: Not Available, Refrigerated : Not Available, Frozen: Not Available 				
GENERAL INFORMATION					
GENERAL INFORM		iot Available, Refrigerated : Not Avai	lable, Frozen: Not Available		

Testing Schedule	Tue, Fri		
Expected TAT	1-5 days		
Clinical Use	Toxoplasma gondii, an obligate intracellular parasite, is an important opportunistic pathogen of immunosuppressed patients. In AIDS patients and transplant patients, this infection may result in a life-threatening encephalitis. T. gondii can also cause a fatal infection of the fetus if an infection is acquired during pregnancy. Fetal death or major abnormalities such as blindness and mental retardation may occur when infection is acquired during the first trimester. PCR methods may be useful in identifying T. gondii in CSF of immunosuppressed patients or in the amniotic fluid of mothers thought to be recently infected.		
Performing Labcorp Test Code	138602		
Notes	Labcorp Test Code: 138602		
CPT Code(s)	87798		
Lab Section	Reference Lab		