

## Digitoxin Level

Order Name: **DIGITOXIN**  
Test Number: 4001800  
Revision Date: 12/12/2022

TEST NAME	METHODOLOGY	LOINC CODE
Digitoxin Level	Chemiluminescence Assay	3559-2

SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	1 mL	Serum	Clot Activator (Red Top, No-Gel)	Refrigerated
Instructions	<p><b>Specimen Type:</b> Red-top tube. <b>Gel-barrier tubes are NOT recommended.</b></p> <p><b>Specimen Storage:</b> SUBMISSION/TRANSPORT (&lt;3 days): Room Temperature. For storage beyond three days, specimen should be refrigerated or frozen.</p> <p><b>Specimen Collection:</b> Serum or plasma should be separated from cells within two hours of venipuncture. Submit serum or plasma in a plastic transport tube.</p> <p><b>Special Instructions:</b> Testing performed at MEDTOX Laboratories, TC 3775</p> <p><b>Specimen Stability:</b> Ambient: 3 days, Refrigerated : 7 days, Frozen: 2 months</p>			

GENERAL INFORMATION	
Expected TAT	2 - 5 days
Performing Labcorp Test Code	790440
Notes	Labcorp Test Code: 790440
CPT Code(s)	80375
Lab Section	Reference Lab