

Chlamydia trachomatis, Neisseria gonorrhoeae Swab - TMA

Order Name: **CT/GC Swab**

Test Number: 6910127

Revision Date: 04/03/2025

TEST NAME	METHODOLOGY	LOINC CODE
Chlamydia trachomatis Swab	Transcription-Mediated Amplification	43304-5
Neisseria gonorrhoeae Swab	Transcription-Mediated Amplification	43305-2
Specimen Source - Prompt	Prompt	

SPECIMEN REQUIREMENTS

Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	Swab	Swab	APTIMA Multitest Swab kit	Refrigerated
Alternate 1	Swab	Swab	APTIMA Unisex Swab Collection kit	Refrigerated

Instructions
Collect vaginal swab in APTIMA Multitest swab collection kit, or endocervical or male urethral swab, rectal and throat specimens in APTIMA Unisex Swab Specimen Collection kit.

Place swab in Swab Specimen Transport Tube, break shaft off at scoreline then recap tube.

Stability: Room Temperature 30 Days, Refrigerated 30 Days, Frozen: N/A

Note: Multiple orders for different collection sites must be placed with collection times at least 1 minute apart to avoid duplication issues.

GENERAL INFORMATION

Testing Schedule	Mon-Fri
Expected TAT	2-3 days
CPT Code(s)	87491, 87591
Lab Section	Molecular Diagnostic