

Volatile Screen

Order Name: **VOLATILES**
Test Number: 4302200
Revision Date: 12/01/2023

TEST NAME	METHODOLOGY	LOINC CODE
Acetone Quant	Gas Chromatography/Mass Spectrometry (GC/MS)	2513-0
Methanol Level	Gas Chromatography/Mass Spectrometry (GC/MS)	n/a
Isopropanol Level	Gas Chromatography/Mass Spectrometry (GC/MS)	n/a

SPECIMEN REQUIREMENTS

Specimen	Specimen Volume (mL)	Specimen Type	Specimen Container	Transport Environment
Preferred	3.0mL (0.5mL)	Serum	Clot Activator (Red Top, No-Gel)	Spin Separate and Freeze
Instructions	<p>Clot Activator (Red Top, No-Gel): Allow to clot then spin, separate serum and freeze ASAP. Specimen Stability: Room temperature: 2 hours, Refrigerated: 28 days, Frozen: 30 months</p> <p>As of 12/1/2023 the Ethanol will no longer be reported in this panel. If needed the Ethanol Screen [4301875] is orderable separately and performed locally, Separate Specimen is required for Ethanol Screen.</p>			

GENERAL INFORMATION

Testing Schedule	Sun - Sat
Expected TAT	24 hours
Notes	Performing Reference Lab: OU Medical Center
CPT Code(s)	84600
Lab Section	Reference Lab