

STD (Sexually Transmitted Disease) Profile

Order Name: **STD Profile**
Test Number: 6906281
Revision Date: 04/20/2020

| TEST NAME | METHODOLOGY | LOINC CODE |
|---|-------------------------------|------------|
| Hepatitis B Surface Antibody, IgG | Chemiluminescence Assay | |
| Hepatitis B Surface Antigen | Chemiluminescence Assay | 5195-3 |
| Hepatitis C Antibody (HCV Ab) | Chemiluminescence Assay | 16128-1 |
| Hepatitis A Antibody (HAV), IgM | Chemiluminescence Assay | 22314-9 |
| Hepatitis B Core IgM Antibody | Chemiluminescence Assay | |
| Herpes Simplex Virus Type 1 and Type 2 Antibodies IgG | Chemiluminescence Assay | |
| Herpes Simplex Virus Type 1/2 IgM IFA | Indirect Fluorescent Antibody | |
| HIV Antigen/Antibody Screen, 4th Gen | Immunoassay (IA) | 56888-1 |

SPECIMEN REQUIREMENTS

| Specimen | Specimen Volume (min) | Specimen Type | Specimen Container | Transport Environment |
|-----------|-----------------------|------------------|--------------------|-----------------------|
| Preferred | See Instructions | See Instructions | See Instructions | Refrigerated |

Instructions

Hepatitis B Surface Antigen and Antibody, Hepatitis C Antibody (HCV Ab); Hepatitis A Antibody (HAV), IgM; Hepatitis B Core IgM Antibody; Herpes Simplex Virus Type 1 and Type 2 Antibodies IgG:
7mL(4mL) Serum Clot Activator SST, Refrigerated

Herpes Simplex 1 and 2 IgM Antibody:
0.5mL(0.1 mL) Serum Clot Activator SST, Refrigerated
Separate from cells ASAP or within 2 hours of collection.

HIV Antigen/Antibody Screen, 4th Gen:
Please submit UN-OPENED Serum Separator tube.
Allow to 15-20 minutes to clot then Centrifuge - Do Not Open!
Specimen Stability: Room temperature: 72 hours, Refrigerated: 5 days, Frozen: 30 days.
Keep Specimens Refrigerated for best stability.

GENERAL INFORMATION

| | |
|-------------------------|---|
| Testing Schedule | Varies |
| Expected TAT | Assay Dependant |
| Notes | <p>HIV-1/2 Antibody Differentiation will be performed at an additional charge (CPT code(s): 86701, 86702). Reflex Order code: HIV-1/2 Antibody Differentiation [3609707] Reflex Result code: HIV 1 Antibody [3609713] LOINC: 68961-2 Reflex Result code: HIV 2 Antibody [3609717] LOINC: 81641-3</p> <p>If HIV-1/2 Antibody Differentiation is Indeterminate or Negative, HIV-1 RNA, Qualitative, TMA will be performed at an additional charge (CPT code(s): 87535). Reflex Order code: HIV-1 RNA, Qualitative TMA [3609719] Reflex Result code: HIV-1 RNA,QL TMA [3609719] LOINC: 25835-0</p> |
| CPT Code(s) | 86706, 87340, 86803, 86709, 86705, 86695x2, 86696x2, 87389 (possilbe HIV reflex testing: 86701, 86702, 87535) |
| Lab Section | Labcorp Oklahoma, Inc. Immunology |