

MySwab Vaginosis Profile + STD - Swab - TMA

Order Name: **MySwab Vag STD**
 Test Number: 6987005
 Revision Date: 04/06/2020

TEST NAME	METHODOLOGY	LOINC CODE
Bacterial Vaginosis - Swab	Transcription-Mediated Amplification	46999-9
Candida Species - Swab	Transcription-Mediated Amplification	47000-5
Candida Glabrata - Swab	Transcription-Mediated Amplification	n/a
Trichomonas Swab (Vaginal)	Transcription-Mediated Amplification	70165-6
Chlamydia trachomatis Swab	Transcription-Mediated Amplification	43304-5
Neisseria gonorrhoeae Swab	Transcription-Mediated Amplification	43305-2

SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	Swab	Swab	APTIMA Multitest Swab kit	Refrigerated
Instructions	Collect Swab in APTIMA Multitest Swab collection kit. Place swab in Swab Specimen Transport Tube, break shaft off at scoreline then recap tube. Stability: Room Temperature 30 Days, Refrigerated 30 Days, Frozen: N/A			

GENERAL INFORMATION	
Testing Schedule	Mon-Fri
Expected TAT	1-2 days after set-up
CPT Code(s)	87801, 87481, 87661 87491, 87591
Lab Section	Molecular Diagnostic