Labcorp Oklahoma, Inc. Test Directory

Human Immunodeficiency Virus 1 (HIV-1) GenoSure® MG Genotyping

Order Name: GenoSure MG Test Number: 5194499 Revision Date: 02/09/2023

TEST NAME			METHODOLOGY	LOINC CODE	
Human Immunodeficiency Virus 1 (HIV-1) GenoSure® MG Genotyping Polymerase Chain Reaction					
SPECIMEN REQUIREM	IENTS				
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment	
Preferred	5 mL	Plasma	EDTA (Lavender Top)	Frozen	
Instructions	Specimen Storage: FREEZE Specimen Collection: Collect Centrifuge specimen within six top tubes should be avoided a DRY ICE UNTIL COMPLETEL frozen. To avoid delays in turn FOR EACH TEST REQUESTI Special Instructions: A sepa	becimen Type: Lavender-top (EDTA) OR plasma preparation tube (PPT(TM)) becimen Storage: FREEZE becimen Collection: Collect specimen in three lavender-top (EDTA) tubes or three PPT(TM) tubes. Do NOT use green-top (heparin) tubes. entrifuge specimen within six hours of collection. Transfer plasma to one or more SCREW-CAP polypropylene transport tube(s). Snap-cap or pop- p tubes should be avoided and may result in specimen rejection. Immediately freeze specimen at -20(degrees) C. DO NOT PLACE SAMPLES ON RY ICE UNTIL COMPLETELY FROZEN. Specimens collected in PPT(TM) need to be centrifuged within six hours and frozen. Ship specimens bozen. To avoid delays in turnaround time when requesting multiple tests on frozen samples, PLEASE SUBMIT SEPARATE FROZEN SPECIMENS OR EACH TEST REQUESTED. becial Instructions: A separate frozen specimen is required for each frozen test ordered. becimen Stability: Ambient: Not Available, Refrigerated : Not Available, Frozen: Not Available			
GENERAL INFORMAT	ION 9 - 15 days				
Clinical Use	Use: Detect resistant protease. Limitations: This pro	Limitations: This procedure may not be successful when the HIV viral load is <500 copies/mL plasma. If there is insufficient virus to produce results, HIV-1 RNA quantitation will be performed to confirm viral load, and the procedure will be billed at the account's prevailing			
Performing Labcorp Te Code	st 551697	551697			
Notes	Labcorp Test Code: 5 Possible Reflex: Order Code: (51945 Result Code: (55853 Result Code: (55853	04) .Reflex HIV Qn RNA PC 185) HIV 1 RNA IU	R		
CPT Code(s)	87900, 87901 (possik	ble 87536 for Viral Load Cor	firmation)		

Lab Section Reference Lab