

NuSwab VG+, HSV

Order Name: **NS VG+, HSV**

Test Number: 5195323

Revision Date: 09/10/2023

TEST NAME	METHODOLOGY	LOINC CODE
NuSwab VG+, HSV	Nucleic acid amplification (NAA)	

SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	1	Swab	APTIMA Multitest Swab -or- Unisex Swab Kit	Room Temperature
Instructions	Container: Aptima® Multitest Swab Specimen Collection Kit or Aptima® Unisex Swab Specimen Collection Kit Collection: Vaginal swab: Collect vaginal fluid sample using the Aptima® swab by contacting the swab to the lower third of the vaginal wall and rotating the swab for 10 to 30 seconds to absorb fluid. Immediately place the swab into the transport tube and carefully break the swab shaft against the side of the tube. Tightly screw on the cap. Causes for Rejection: Specimen with incorrect patient identification; unlabeled specimen; inappropriate specimen transport conditions, including specimens received frozen; specimens received after prolonged delay (usually >72 hours); specimen leaked in transit; specimen in expired transport or incorrect transport device; specimens with inappropriate source for test requested; specimen with fixative or additives; Aptima® urine transport; Aptima® swab transport >30 days from collection; Aptima® swab specimen without a swab; cleaning swab (white-shaft swab) in Aptima® swab transport; any non-Aptima® swab submitted in Aptima® transport device; transport device with multiple swabs; bloody or grossly mucoid specimens; bacterial swabs; specimen in ProbeTec™ UPT transport; ProbeTec™ Q-swabs Storage Instructions: Maintain specimen at room temperature or refrigerate (2°C to 30°C). Stability Requirements: Room temperature 30 days, Refrigerated 30 days, Frozen Not acceptable.			

GENERAL INFORMATION		
Expected TAT	3-4 days	
Performing Labcorp Test Code	180066	
Notes	RESULT CODE	RESULT NAME
	5195307	Atopobium vaginae
	5195308	BVAB 2
	5195309	Megasphaera 1
	5195303	Candida albicans, NAA
	6987011	Candida glabrata, NAA
	6910157	Trich vag by NAA
	6910149	Chlamydia trachomatis, NAA
	6910153	Neisseria gonorrhoeae, NAA
	6987013	HSV 1 NAA
	6987014	HSV 2 NAA
	LOINC	
CPT Code(s)	87591, 87661, 87798x3, 87529x2, 87491, 87801	
Lab Section	Reference Lab	